



## **How to Use Fentanyl Test Strips**

### **Get a Good Sample of Your Drugs**

If testing powder, shake the bag to make sure all powder is thoroughly mixed. If drugs aren't powder, crush the whole pill or crystal until you have a powder and then mix or shake thoroughly. You only have to test each bag, batch, or pill once.

### **Sample Size**

You will want to use at least a 10mg sample of your drugs. 10mg would be the equivalent of Abraham Lincoln's hair on a penny, or approximately the size of the circumference of an eraser on a standard pencil. You can also just use your bag residue.

### **For Meth or MDMA**

Use 5cc, or one teaspoon, of water for every 10mg of drugs.

One of the included sterile waters is 5cc.

Use the whole thing.

Mix with your 10mg of drugs in a cooker or other clean container.

Do not stir it with the test strip, use something else like a capped syringe.

Only test with each cooker once, or rinse thoroughly between tests.

### **For Heroin**

Use 2.5cc, or a half a teaspoon, of water for every 10mg of drugs.

One of the included sterile waters is 5cc.

Use half of it.

Mix with your 10mg of drugs in a cooker or other clean container.

Do not stir it with the test strip, use something else like a capped syringe.

Only test with each cooker once, or rinse thoroughly between tests.

## **Use the Strip**

1. Dip the wavy end of the strip into the sample in the cooker.
2. Let the strip absorb the water for 15 seconds.
3. Take the strip out of the sample and place it on a flat surface for two minutes.
4. Read the results with the information provided below.

## **Reading Your Results**

Test lines will be red.

One line is Positive, meaning there was fentanyl detected in the sample. The darkness of the line does not mean anything, as the strip does not measure fentanyl potency.

Two lines is Negative, there wasn't any fentanyl detected, however false negatives are possible due to the variety of analogs. Still proceed with caution.

No lines, or no top line, means the test is Invalid and should be done again.

## **You can get Fentanyl Test Kits at NorthPoint Harm Reduction & Testing Services located at:**

710 W Broadway Ave  
Minneapolis, MN 55411

### **Our hours are:**

Tuesday and Wednesday 2-5pm  
Thursday and Friday 11am-5pm

### **You can call or text us at:**

612-267-0305 or 612-223-3682